

## Usage instructions for Pressure Pad, plastic Pad I

The middle strap pad 'Pressure Pad, plastic Pad I' is used as an accessory to be used at the back of the foot in combination with a foot support/orthotic shoe in order to improve pressure distribution and avoid pressure points and constrictions.

**PAD I, Size M: MP-2201-001**

BASIS-UDI: 426020155ZBHMRP5B


### Important information:

**Storage:** Protected from frost, light and excess humidity

**Cleaning:** dishwasher-safe, do not use harsh disinfectants.

**Disposal:** with household waste

**Date of manufacture:**

For packaging with this symbol: 

**Service life:** 8 years

**Please note:** Risk of suffocation!

**The product should only be inserted after consultation with an orthopaedic technician/doctor and on the basis of their assessment.**



The foot support pull/attachment strap should be inserted into the wide strap guide in the central strap pad, which threads it onto the strap.



Once the central strap pad is correctly inserted into the shoe, its position on the back of the foot should be as form-fitting as possible.

Overlaps that could cause pressure points should be avoided.  
The Pressure Pad should only be used on feet if their condition allows it.



Sameda GmbH, Am Petersberg 36,  
D-29389 Bad Bodenteich, Germany

Mail: [info@semada.de](mailto:info@semada.de)  
Web: [www.semada.de](http://www.semada.de)

